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Read this package insert carefully before using this product

# OTIX

## Ear drops and ear spray

### FLORIS

OTIX drops are emollient hygienic drops intended for cleaning ear canal by dissolving excess earwax.

OTIX also helps relieve pain and inflammation by coating and isolating the affected area.

OTIX eardrops help remove wax plugs and creating an environment that encourages the healing process from ear infections related to eczema, dermatitis or mycosis.

#### Active ingredients:

Olive oil, Caprylic/ Capric triglycerides, Squalane, Benzyl alcohol, Thymol, Geranium oil, Tea tree oil.

#### Inactive ingredients:

PEG 20 Glyceryl triisostearate, Bisabolol.

#### Cerumen -naturally secreted earwax:

The ear is a complex organ consisting of three distinct compartments: external, middle and inner ear. Each one of the compartments enables a person to hear. The external auditory canal, which links the outer ear to the ear drum, is lined with sebaceous glands which naturally secrete a yellow greasy wax substance, called earwax – cerumen.

#### Blockage and impaction:

Smaller or oddly shaped ear canals make the natural process difficult and lead to wax impactions. Improper use of cotton tipped swabs (pushing the rest of the wax deeper into the ear canal) and wearing ear aids or earplugs increases the occurrence of earwax blockage.

#### Complications of earwax:

Untreated impacted earwax can lead to ear infection (otitis externa & otitis media), ear pain and sometimes light bleeding. Elderly people with diabetes and people with weak immune system are most likely to get affected. Symptoms of earwax impaction include: irritation, sensation of inflammation, itching or drainage of the ear canal, plugged or stuffed sensation, ear pain and decreased hearing (consider: hearing loss).

#### How does OTIX work?

OTIX ear drops offer an effective relief through a three-way action:

- Olive oil coats, soothes and isolates the affected area while assisting in the softening of wax for easier removal.
- Caprylic / Capric Triglycerides and Squalene are lipophilic solvents that solubilize and absorb moisture from exposed and inflamed areas relieving pressure in the middle ear.
- Herbal extracts and volatile essential oils assist in relief of ear pain and ear infection.

#### Indication:

OTIX eardrops are for ear hygiene, for dissolving cerumen or plugs of wax, and helps to ease ear pain and assist in the treatment of ear inflammation or ear infection.

#### Directions/ Instructions for use:

##### • Removal of wax:

Squeeze 3-4 OTIX drops into the ear and then plug with cotton wool. In cases of hard wax plugs, repeat the treatment three times a day for 2-3 days. Avoid direct contact of dropper with ear.

##### • Swimmer's Ear and Otitis Media:

Tilt the head to one side with infected ear facing upwards, squeeze into the canal 3-4 OTIX drops, and plug ear with cotton wool. Repeat every two hours until pain and congestion are relieved.

#### Precautions of use: For external use only.

OTIX should not be used for bleeding ears, or for perforated eardrum.

Also, do not put any foreign object in the ear canal.

Keep away from children.

Do not apply on eyes.

Do not swallow.

Do not use if you are aware of any allergy or sensitivity to one of the ingredients. Consult with your doctor if the condition does not improve.

Warnings: if you experience a severe spinning sensation or loss of balance, persistent vomiting, high fever or a sudden loss of hearing go to the hospital.

For children under the age of 3 years, please consult your pediatrician.

The product must be used under adult supervision.

In case of eye contact, wash thoroughly with running water for several minutes.

In case of deterioration, please contact a Medical Care Center or an Emergency Room immediately and bring the product's package with you.

#### Storage:

Keep in a dry cool place below 25°C. Use before the expiry date printed on label.

With approval of Ministry of Health, Medical Device Division, registration N.: 6650401

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